

kitett®

FISIO breast pump with automatic and adjustable suction cycle

FISIO gives mother the natural feel of breastfeeding by faithfully and smoothly reproducing the action of a breastfeeding baby.

FISIO offers comfort and the opportunity to extend breastfeeding by combining simplicity of use and easy maintenance.

EASY TO USE

- A single adjustment control
- Adjustable suction cycle synchronized with suction strength
- Gentle stimulation and expression phases - according to the physiology of breastfeeding

EASY TO MAINTAIN

- Reliable
- Easy to assemble and clean

OPTIMUM COMFORT

- Customizing settings
- Used with the KOLOR® EXPRESSION KIT, single or dual pumping

DUAL PUMPING

- Time saving
- Increased levels of prolactin
- A significant increase in the prolactin rate: increased lactation

**Extend breastfeeding
with serenity**



A single control
for adjustment

CE 0459 - 2003

BREASTFEEDING BY DTF MEDICAL



kitett®

KOLOR® Expression Kit



One-piece, soft, multi-size and fun breast shield.

COMFORTABLE



- Ergonomic and flexible shield
- Soft contact on the skin

EASY TO CLEAN

- The gasket cannot be lost
- Comfort shield fixed on the funnel

MULTI-SIZE

- 2 comfort shapes: S (Small) and L (Large)
- 4 inner diameters:

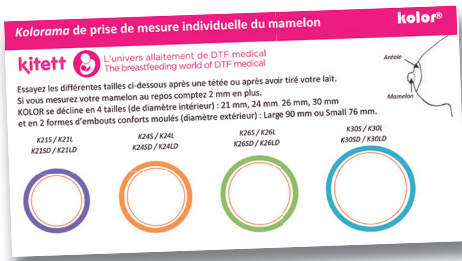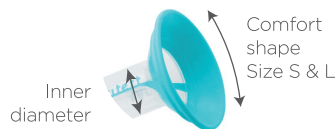

Size Guide available on request



EXPRESSION KIT
SINGLE or DOUBLE



0459 - 2011

0113.DOCCOM.NOTFISIO.GB.005 - Documentation destinée aux Professionnels de Santé. Photographies non contractuelles.

BREASTFEEDING BY DTF MEDICAL



Diffusion Technique Française 19, rue de la Presse - BP 132
42003 Saint-Etienne Cedex 1 - France

Tél. : (33) 4 77 74 51 11 - Fax : (33) 4 77 79 67 72

E-mail : dtf@dtf.fr - www.dtf.fr




THE NEW EXPRESSION KIT by **kitett**

kolor[®] ONE-PIECE, SOFT, MULTI-SIZE AND FUN BREAST SHIELD

kolor[®] *in details.*

The new **kolor**[®] Expression Kit is a result of 2 years' research and development and takes into account the recommendations and expectations of mothers and breastfeeding specialists.

Characteristics:

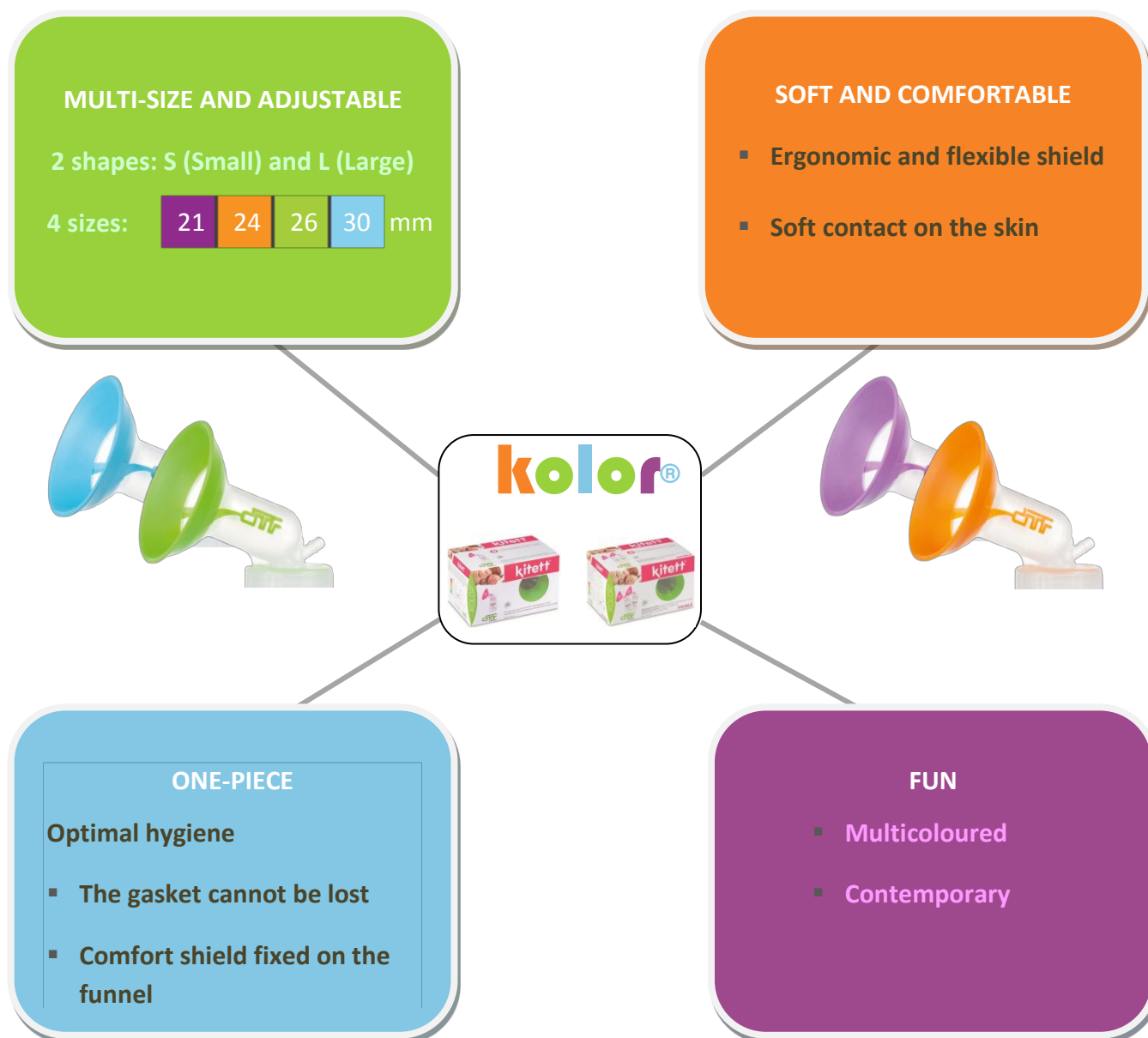

Discover **kolor**[®] on our website www.dtf.fr

The **kolor**[®] Expression Kit is designed exclusively for KITETT automatic and FISIO breast pumps.




Bi-material breast shield:

- This original Expression Kit combines rigid and soft materials for better efficiency and comfort.

The best comfort for mothers:

- 0% Bisphenol A (BPA),
- 4 inner diameters (21, 24, 26, 30 mm) corresponding to the nipple size,
- 2 comfort shapes (Small and Large) according to the breast size,
- A “Kolor-ruler” to choose the right size.

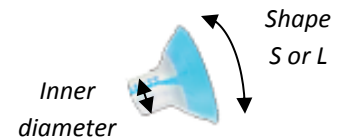

Easy to clean/sterilize options:

- Cleaning with soapy warm water, then rinsing carefully
- Sterilizing: Boiling water for 10 min;
Baby bottle steam sterilizer, except tubing which should briefly be dipped in boiling water and then thoroughly dried;
Dishwasher.

Dimensions and weight:

KOLOR Small: Breast shield diameter: 76 mm Weight: 21 g

KOLOR Large: Breast shield diameter: 90 mm Weight: 30 g

References:

Each KOLOR EXPRESSION KIT comprises 1 KOLOR breast shield, 1 bottle, 1 connecting tube (two of each for DOUBLE KOLOR EXPRESSION KIT)

Inner Ø in mm	SINGLE Expression Kit		DOUBLE Expression Kit	
	Kolor Small breast shield	Kolor Large breast shield	Kolor Small breast shield	Kolor Large breast shield
21	K21S	K21L	K21SD	K21LD
24	K24S	K24L	K24SD	K24LD
26	K26S	K26L	K26SD	K26LD
30	K30S	K30L	K30SD	K30LD

KOLOR breast shield only: Reference WB21S/WB21L – WB24S/WB24L - WB26S/WB26L or WB30S/WB30L

Photographies non contractuelles

Made in France by



diffusion technique française
19, rue de la Presse 42000 SAINT-ETIENNE France
Tél. +33 (0)4 77 74 51 11 Web : www.dtf.fr

AB/SS 01/2012

MAJ GC/CC/MPS 03/2013